Components for the TMI Single Upright Patient Helper

ITEM # PRODUCT

6337 TMI Single Upright Patient
Helper Complete

6338 TMI Support Bracket

6337-3 3pc Bar Set; 46"Upright,Double
Curve Bar, Octagonal Bar

6040 Trapeze Assembly

This Patient helper is for use with the Head

End Adapter from the bed manufacturer

CONTACT OUR OFFICE FOR MORE INFORMATION

- * All TMI patient helpers are designed to assist patient mobility. They are not designed to suspend a patient off the bed.
- **Check trapeze chain daily. Make sure it is free from kinks before use.
- ***Do not twist or tighten the trapeze chain during use.





TEXAS MEDICAL INDUSTRIES, INC.

TEXAS MEDICAL INDUSTRIES, INC. 1409 INDUSTRIAL DR. ROYSE CITY, TX 75189

Phone: 1-800-332-9556 Fax: 1-800-636-1864

Email: danielle@tmiunited.com Web: www.tmiunited.com



TMI Single Upright Patient Helper #6337



TMI

Manufacturers of Quality Orthopaedic & Orthotic Products for Modern Techniques Since 1962

PHONE: 1-800-332-9556 www.tmiunited.com




Installation Instructions for the TMI Single Upright Patient Helper #6337

Application is as follows:

- Attach horseshoe shaped bed adapter to head of bed. (Available from Bed Manufacturer)
- 2. Attach support bracket (#6338) to bed adapter using bolts provided.
- 3. Place upright into the support bracket and secure with support bolt provided.
- 4. Slide octagonal bar into double curve bar forming the overhead extension. Secure with bolt provided.
- 5. Insert curved bar section into upright. Tighten swivel security knob to secure.
- 6. Suspend trapeze assembly from the octagonal bar.
- Adjust all knobs and the trapeze to position desired. Tighten all knobs by hand.

This unit consists of a single upright composed of three sections with the trapeze attaching to the third section made of octagonal tubing. The remaining two sections are of coated steel with adjustable swivel action in the double curve bar. The upright slides into a support bracket constructed of coated steel for added strength. The support bracket is attached to a bed adapter you obtain from the bed manufacturer. All knobs should be security checked several times a day.

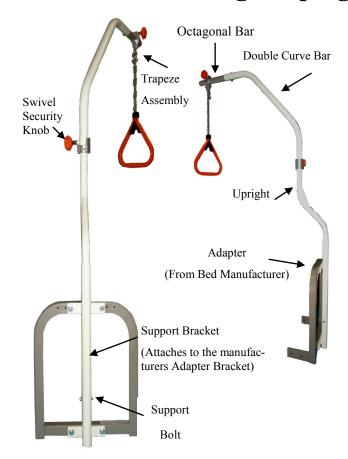

Maintenance:

Check and hand tighten knobs each time the patient is visited. Check all screws and clamps for wear and replace as needed. Unit may be cleaned with standard Hospital cleaning agents. *Do not bleach*.

*Maximum weight capacity is 290lbs patient weight. When properly installed and maintained.



FDA REGISTERED - MADE IN THE U.S.A



TEXAS MEDICAL INDUSTRIES, INC.

TEXAS MEDICAL INDUSTRIES, INC. 1409 INDUSTRIAL DR. ROYSE CITY. TX 75189 Phone: 1-800-332-9556 Local: 972-636-9556

Fax: 1-800-636-1864

^{*} All TMI patient helpers are designed to assist patient mobility. They are not designed to suspend a patient off the bed.

^{**}Check trapeze chain daily. Make sure it is free from kinks before use.

^{***}Do not twist or tighten the trapeze chain during use.